

General Information

The traditional reusable Spackman Cannula is liked by clinicians but gives great difficulty to Sterile Services who have to try and clean a long narrow cannula with a closed end. Concerns over cleanliness and the associated risk of post-operative infection have led to demand for a single use instrument.

SUS have taken the reusable instrument and matched the geometry for equivalent uterine manipulation capability, but improved the clamp action. As the traditional instrument it has a well rounded distal end for easy placement through the endocervical canal and transverse holes at the distal end for even dye distribution. The clamp incorporates reinforced hooks for tenaculum traction and the handle accepts a standard luer for injection dye into the uterine cavity.

The main features and benefits of the S029 are:

- Single use instrument – reduced risk
- No clamp screw to drop or lose
- Good seal on cervix
- Latex free and DEHP free
- Quick and easy to insert and remove

How to Use

Insert cannula

- 1 Apply tenaculum/volsellum, pull clamp back and engage clamp hooks in handle
- 2 Pull clamp back without touching the lever (self-locking) to ensure good seal
- 3 To release tenaculum/volsellum, press lever to slide clamp to distal end

